# JAY<sup>®</sup>

*Jay*<sup>®</sup> J-Gel Cushion  
Instruction Guide

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**CAUTION**

Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

The J-Gel cushion is designed to provide sitting comfort and postural support as well as to help reduce the incidence of skin breakdown. However, no cushion can completely eliminate sitting pressure or prevent skin breakdown. The J-Gel cushion is not a substitute for good skin care including: proper diet, cleanliness and regular weight shifts/ pressure reliefs. In addition, prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing.

Users should ensure their skin is inspected daily for signs of skin breakdown. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and consult your doctor or therapist. If you require a higher pressure-relieving product for skin protection, ask your Sunrise Medical- Jay supplier for a free demonstration of any of the other Jay cushion products.

Do not leave the cushion outside overnight at temperatures below 40°F (5°C). Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.

Avoid exposure to direct sunlight.

Avoid contact with any sharp objects or pointed edges that may tear or puncture the gel.

The J-Gel Cushion is free of natural rubber latex.

**NOTE**– To ensure the proper use of your cushion, please read these instructions and keep them for future reference.

**FITTING A J-GEL CUSHION**

A cushion should ideally be fitted to you by a health care professional. Place the cushion to the back of the chair with the material identification tags to the back, the contoured side facing up and the “bump” towards the front. The product label should be visible on the right side of the cushion. Adjust the footplates so your legs rest comfortably on the cushion. Lowering the footrests on your wheelchair may help to reduce the pressure on your ischial tuberosities (seat bones) and on your coccyx (tailbone).

**NOTE**– Proper foot adjustment may enhance sitting comfort. Also, avoid using the cushion on overly stretched seat upholstery. Replace stretched seat upholstery or purchase a Jay Solid seat insert for your cushion. Avoid using overly stretched backrest upholstery. Stretched upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched backrest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact a Sunrise Medical- Jay supplier for a free demonstration of these products.

**EASY MAINTENANCE**

Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended. Check the covers for tears and excessive wear.

**To Clean Cover**

Remove the cover from the base. Turn the cover inside out and machine wash in warm water and drip or tumble dry on low heat. No ironing is required.

**NOTE–** Do not dry clean covers or use industrial washers and dryers to clean the cover. Do not steam autoclave. Do not bleach.

After washing, fit cover back over cushion base and gel pad. Ensure the back of the cover matches up to the back of the foam base and gel pad assembly.

**NOTE–** If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.

Use only Jay-designed covers in conjunction with the J-Gel Cushion.

**To Clean Foam Base**

Remove the cover and gel pad from the foam base. Wipe with warm water and soap. Wipe off with a clean cloth.

**NOTE–** Avoid submersion of the foam base in water.

Reinstall the gel pad, then fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base and gel pad assembly.

**To Clean Gel Pad Overlay**

Remove the cover and gel pad from the foam base. Recommend use of a 1:10 solution of bleach and water to clean the gel cube pad surfaces. You may also use any conventional hospital approved topical equipment cleaners and disinfectants that do not contain alcohol. (Never clean any surfaces of cushion with alcohol). Wipe off with a clean cloth. The gel overlay may also be cleaned with warm water and soap. Autoclaving and gas sterilization are not recommended. Always thoroughly rinse gel pad with clear water before placing back into service. This helps eliminate the possibility of skin irritation due to cleaning solution residue.

**Reassembling the J-Gel Cushion After Cleaning**

Make sure cover, foam base and gel pad overlay are completely dry before reassembling. Orient the front and back of each component per the labels on each, then attach the gel pad overlay using the Velcro® strips as location indicators. The gel pad should fit symmetrically over the foam base. Lastly, replace cover over entire assembly. Ensure the back of the cover matches up to the back of the cushion/gel assembly.

**Repairs**

Manufacturer offers a repair kit with instructions for the Gel cube pad.

**LIMITED WARRANTY**

Sunrise Medical guarantees its products to be free from defects in materials and workmanship for a specific period of time from the date of purchase.

- The J-Gel Cushion is guaranteed for 2 years from date of purchase.

Sunrise Medical will, at its option, repair or replace your cushion without charge. The warranty does not apply to punctures, tears or burns, or to the cushion cover.

Claims and repairs should be processed through the nearest authorized Sunrise Medical– Jay equipment supplier. Except for express warranties made herein, all other warranties including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are no warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

Maximum user weight:350 lbs (158 kg)

**NOTE–**

The J-Gel Pad overlay is intended to be used in conjunction with the J-Gel foam base. Use of the J-Gel pad overlay without the J-Gel foam base will impede the use for which the cushion is intended. Use of the J-Gel pad overlay without the J-Gel foam base will void the warranty and the manufacturer assumes no liability.

**REPAIR KIT M4204 INSTRUCTIONS:**

1. Area to be repaired must be clean and dry before applying repair tape.
2. Any sticky surface exposed or damaged may be cleaned with mineral oil. Remove mineral oil residue using warm, soapy water. Dry surface completely before applying tape.
3. Measure damaged area and cut repair tape to size. Allow for 6 mm overlap on all sides.
4. Carefully peel back approximately 12 mm of brown backing paper and crease it.
5. Carefully apply exposed portion of clear tape to damaged surface. Push out any air bubbles which may be trapped beneath the tape.
6. Continue to carefully apply repair tape while slowly removing paper backing. Push out any air bubbles.

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**Sunrise Medical GmbH**
**Sunrise Medical B.V.**
**Sunrise Medical AB**