Owner's Manual

4000 Series

Mobile commode & shower chairs

IMPORTANT CONSUMER INFORMATION

- NOTICE:** This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.
- SUPPLIER:** This manual must be given to the user of this product.
- USER:** Before using this product, read this entire manual and save for future use.

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ATTENTION! PLEASE READ!

Please review the **LIMITED WARRANTY** included with these instructions. If the warranty terms are not acceptable, contact Nuprodx, Inc. for information about returning the chair for a refund (restocking fees and shipping charges will apply). This chair is designed and intended for use by a single occupant solely as an aid for showering or bathing, going to the toilet, or for indoor transfer. Any other use, including outdoor use, is strictly prohibited and will void the warranty.

OWNER'S MANUAL

4000

INDEX

INTRODUCTION	03
PARTS SUPPLIED	03
GENERAL WARNINGS	04
SETTING UP	04
FINAL CHECK	06
ADJUSTMENTS	06
INSTRUCTIONS FOR SAFE USE	07
OPERATING INSTRUCTIONS	08
TX PACKING INSTRUCTIONS	08
STORAGE & TRANSPORTATION	08
CLEANING & MAINTENANCE	08
SAFETY CHECKLIST	09
LIFE EXPECTANCY	09
DISPOSAL	09
LIMITED WARRANTY	09
FIGURES & NOTES	30-31

INTRODUCTION

The Nuprodx MultiChair 4000 Series has been designed to provide safe, reliable, and adaptable support for individuals who require assistance with bathing and toileting. This modular rolling shower and commode platform includes both caregiver-assist models and self-propelled variants, allowing the chair to be matched to the user's mobility needs and level of independence.

The standard rolling models are intended for situations where a caregiver positions and maneuvers the chair, offering secure mobility and easy access to the toilet or shower without complex transfer steps. The self-propelled models, equipped with larger rear wheels, allow users with sufficient upper-body function to propel and position themselves independently for bathing or toileting tasks. Across all versions, the MultiChair 4000 Series promotes safety, comfort, and hygiene in both home and clinical settings while reducing the physical demands on caregivers.

SERIAL NUMBER & MAX WEIGHT

The serial number and model information label is located on the underside of the seat frame.

MC4000 MC4000Ped	300 lbs (136 kg)
MC4200	400 lbs (181 kg)
MC4000Tilt MC4000TiltPed	300 lbs (136 kg)
MC4200Tilt	400 lbs (181 kg)
MC4020 MC4020RX MC4020Ped	300 lbs (136 kg)
MC4220 MC4220RX	400 lbs (181 kg)
MC4024	300 lbs (136 kg)
MC4224	400 lbs (181 kg)
MC4000TX	300 lbs (136 kg)
MC4000Leckey	160 lbs (73 kg)

Cantilever Arms: arms will be installed on backrest posts

Swing-Away Arms: arms included to install on seat frame

- (1) headrest mounting bracket (if ordering tilt or headrest)

HEADREST ASSEMBLY (if ordering tilt or headrest)

- (1) headrest post
- (1) headrest pad

FOOTREST ASSEMBLIES

(2) footrest assemblies (left and right)

Adjustable Swing-Away Footrests

- (2) adjustable swing-away footrest assemblies (left and right)
- (2) footrest clamps (pre-mounted on front legs)

Seat-Mounted Footrests

(2) seat-mounted footrest assemblies (left and right)

HARDWARE & TOOLS

- (1) 1/4" yellow T-handled Allen wrench (included for assembly)
- (1) 5/32" Allen wrench (included for assembly)
- (1) 1/8" Allen wrench (if ordering headrest)

LECKEY MODELS

- (1) Leckey seat frame with mesh cover
- (1) Leckey chest and hip positioning belt pre-installed

TRAVEL MODELS

(1) carrying case with handle and casters

PARTS SUPPLIED

SEAT FRAME

Includes commode seat (installed on tilt assembly if applicable) mounted to the frame

LEG ASSEMBLIES

STANDARD MODELS

(4) pinned offset straight legs with casters (5 or 6")

TILT MODELS

- (2) front pinned offset straight legs with casters (5 or 6")
- (2) rear 90-degree bent legs with casters (5 or 6")

SELF-PROPELLED MODELS

- (2) pinned offset straight legs with casters (5 or 6")
- (2) pinned offset straight legs with axle blocks for wheels

BACK ASSEMBLY

- (2) backrest posts
- (1) backrest

Padded Backs: back will be installed on backrests posts

Cloth Backs: back included to wrap around backrest posts

- (2) armrest assemblies (left and right)

WARNING

The word “WARNING” refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons.

GENERAL WARNINGS

NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact Carrier/Nuprodx, Inc. for further instructions.

WARNING

DO NOT install this equipment without first reading and understanding the manual. If you are unable to understand the Warnings and Instructions, contact a qualified clinician or supplier - otherwise serious injury or damage may occur.

WARNING

Use only as intended. The MultiChair 4000 Series is designed and intended for use by a single occupant solely as an aid for showering or bathing, going to the toilet, or for indoor transfer. Any other use, including outdoor use, is strictly prohibited.

WARNING

A qualified attendant or caregiver must be present during all transfers and while the user is seated in the chair, unless the user has been specifically assessed as capable of independent operation.

WARNING

Hook and Loop Velcro should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the Velcro.

WARNING

Bolts, knobs, and quick-release pins should be tightened before each use. Loose hardware can cause instability or part failure.

WARNING

Only use Nuprodx-approved components and accessories. Unauthorized modifications may result in injury or void the product warranty.

WARNING

The chair is designed for wet environments, but water can make surfaces slippery. Always maintain control of the chair when near a bathtub, shower, or toilet.

WARNING

Examine the frame, casters, and locking mechanisms for wear, cracks, or corrosion. Discontinue use immediately if any damage is found and contact Nuprodx Inc. for service.

WARNING

Keep hands and clothing clear of pinch points when folding armrests, inserting legs into sockets, or adjusting moving parts

SETTING UP

UNPACK & PREPARE

1. Remove all parts from the box.
2. Remove all tape and/or packing from the parts. Keep shipping materials for future use and away from children and pets.
3. Lay out each component to ensure all parts are present.
4. Keep the yellow T-handle Allen wrench handy - it will be used for all tightening steps.

OPEN SEAT FRAME

FOR TX MODELS ONLY

Open the seat frame, lock the slide bolts, and position the frame upside down. **(REFER TO FIGURE 1)**

WARNING

Failure to ensure that the slide bolts are correctly positioned and locked into place will cause the chair to become unstable. This could cause serious injury or even death!

ASSEMBLE THE SEAT FRAME

1. Identify the seat frame
 - A. This is the frame that includes the commode seat (with tilt system if applicable).
 - B. Place the frame upside down on a protected surface to prevent cushion damage.
2. Install the (4) caster legs
 - A. Each leg has an offset elbow. The offset must face outward from the center of the frame. If ordering tilt, the rear, 90-degree bent legs will not have an offset elbow.
 - B. Identify the front legs. These have footrest clamps pre-installed if ordering adjustable swing-away footrests. The front legs must be installed on the front side of the frame.
 - C. Identify the rear legs. If ordering tilt or Leckey, these are the 90-degree bent legs.
 - D. Insert each leg into the corresponding socket until the stainless-steel pin clicks into the frame hole. There are 3 frame holes for installing straight legs at different angles (0, 45, and 90°). 45° is the ideal starting position. 0 and 90° may be used for increasing or decreasing the overall width. **(SEE FIGURE 2)**
 - E. The top end of each leg should fit easily into the socket of the frame.
3. Secure the legs
 - A. Using the yellow T-handle Allen wrench, tighten the corner bolts at all four mounting points.
 - B. Do not overtighten; bolts should be snug but not forcefully torqued.
4. Verify installation
 - A. Ensure each leg is fully seated, and all elbows angle evenly outward.
 - B. Once secure, flip the seat frame upright.
 - C. Gently rock the frame to confirm stability.

WARNING

If any of the legs are not fitted correctly into the socket of the seat frame or are not securely tightened there is a risk of the

legs dislodging completely from the chair. This could cause serious injury or even death!

INSTALL THE LECKEY CHAIR

1. The front side of the Leckey frame has two mounting slots on each side, and the back side of the frame has one mounting slot on each side.
2. Pick up the Leckey chair and slide it in between the brackets of the Nuprodx frame. Be sure the mounting points align with the holes on the sides of the Leckey chair. **(SEE FIGURE 3)**
3. Use the provided hardware to secure the Leckey chair in its mounting brackets. Slide the bolt with the washer all the way through the bracket and chair from the outside.
4. Secure the bolt by threading the nut onto the opposite end of the bolt from the inside. **(SEE FIGURE 4)**
5. While holding onto the nut, use the yellow T-handle Allen wrench to tighten the chair down to the frame.
6. Repeat for all four corners.

INSTALL THE BACKREST ASSEMBLY

1. Identify the backrest assembly
This is the frame that includes 2 backrest posts with the backrest and armrests attached.
2. Insert the backrest posts
For Padded Backs: The backrest pad should face forward, toward the seat cushion and commode opening.
For Cloth Backs: The back posts bend outwards, away from the seat cushion and commode opening.
 - A. Insert each backrest post into the rear mounting holes of the seat frame. **(SEE FIGURE 5)**
 - B. If backrest posts do not easily slide into the mounting holes, slightly loosen the internal clamp bolts on the bottom of each backrest post while inserting them into the mounting holes. Do not remove the internal clamp bolts completely.
 - C. Ensure both posts are fully inserted and the bottom indexing tab on the back-posts fit into the matching indexing features on the mating seat frame. The backrest will sit level.
3. Tighten the clamps
 - A. Using the yellow T-handle Allen wrench, locate the internal clamp bolts on the bottom of the back posts and the rear corners of the seat frame.
 - B. Tighten each clamp evenly to secure the backrest posts firmly in place.
 - C. Avoid overtightening; ensure the posts remain vertical and aligned.**For Padded Backs:** Also tighten the four button head capscrews that attach the back cushion to the back posts.
4. Verify installation
 - A. Gently push and pull on the backrest to ensure it does not move.
 - B. Confirm that both armrest clamps (if attached) remain level and secure.

WARNING

Both backrest posts must be completely engaged with the seat frame and securely tightened. If they are not fitted correctly, there is a risk of the posts dislodging from the seat frame. This could cause serious injury or even death!

INSTALL THE CLOTH BACK

1. Wrap the cloth back around the back posts and secure the 3 rear adjustable straps using the clips.
2. To prevent the cloth back from moving up and down the backposts, you may utilize the included Velcro strips by cutting the included Velcro strips to the desired length, attaching them to the back posts, and pressing the cloth back against the Velcro.

NOTE: Back tension adjustments are made possible by changing the length of the straps.

WARNING

Tension adjustments made to the cloth back should never be performed with the user in the chair. Make sure each strap on the cloth back is securely locked before use. Ignoring this safety precaution may cause the cloth back to come loose and possibly separate from the backrest posts, resulting in serious injury or even death!

INSTALL THE PUSH HANDLES

1. Position each push-handle at the top of each back post.
2. Press down on the brass buttons while sliding the push-handles into the back posts. **(SEE FIGURE 6)**

INSTALL THE HEADREST ASSEMBLY

1. The headrest clamp will be installed on the backrest assembly.
2. Use the yellow T-handle Allen wrench to loosen the two bolts on the headrest bracket.
3. Slide the extension tube with the headrest attached through the top of the headrest clamp.
4. Use the yellow T-handle Allen wrench to tighten the two bolts on the headrest bracket to secure the headrest in place.

INSTALL THE ARMREST ASSEMBLIES

Cantilever Armrests

These may come pre-installed on the backrest assembly. If not:

1. Remove the two bolts holding the armrest clamp together.
2. Take each arm with half of the clamp and position it onto the correct side of the corresponding back post to the desired height.
3. Take the second half of the clamp and position it inside the back post to secure the arm in place.
4. Tighten the armrest clamp with the included yellow T-handle Allen wrench. **(SEE FIGURE 7)**

Swing-Away Armrests

1. Slip the end of the arm with the steel pin into the matching holes in the frame.
2. Lower down the arms until the latch engages around the black plastic ball attached to either side of the frame.

WARNING

The armrests are not designed to support the entire body weight of the user. Using the arms as a support for transferring into the chair and/or putting extreme amounts of pressure on them could result in damaging the arms or a fall and is not recommended. This could result in serious injury or even death!

INSTALL THE FOOTREST ASSEMBLIES

Adjustable Swing-Away Footrests

1. Locate the footrest components
 - A. Identify the left and right footrest assemblies.
 - B. Locate the footrest clamps pre-installed on the front legs of the seat frame. Verify that the clamps are positioned forward.
2. Install the footrests
 - A. Insert each footrest tube into its corresponding clamp on the front legs.
 - B. The footplates should face towards each other and level when installed correctly. **(SEE FIGURE 8)**
3. Adjust the clamp orientation and position by loosening the clamps and repositioning. Tighten the clamps after repositioning. Do not overtighten; bolts should be snug but not forcefully torqued.
4. Adjust the height of each footplate so it aligns with the user's leg position.

Seat-Mounted Footrests

1. Locate the footrest components
 - A. Identify the left and right footrest assemblies.
 - B. Locate the footrest mounting holes on the front corners of the seat frame.
2. Install the footrests
 - A. Slide the footrests into the front corners of the seat frame and tighten the corner clamps. Do not overtighten; bolts should be snug but not forcefully torqued.

Verify Installation

Check that both footrests are firmly locked in position and do not rotate or slip when light downward pressure is applied.

WARNING

The footrests are not designed to support the entire body weight of the user. Using the footrests as a support for transferring into the chair and/or putting extreme amounts of pressure on them could result in damaging the footrests or a fall and is not recommended. This could result in serious injury or even death!

FINAL CHECK

Before operating the MultiChair 4000 Series, carefully verify that the chair is fully assembled, properly adjusted, and safe for the intended user.

1. Verify all hardware and connections

- A. Ensure all bolts and clamps are tightened securely using the appropriate tools.
- B. Confirm that the brass height-adjustment button on each leg is fully engaged and visible.

2. Check frame stability & alignment

- A. Confirm that the chair rests evenly on the floor without rocking.
- B. Ensure the seat frame, backrest, and armrests are aligned and do not shift under gentle pressure.

- C. Verify that all leg heights remain consistent after adjustments.

3. Inspect functional components

- A. Check that the seat cushion, armrests, and footrests are properly attached and firmly positioned.
- B. Move armrests and footrests through their full range to confirm unobstructed operation.
- C. Confirm the tilt mechanism locks safely before transferring a user (if equipped).

4. Perform an unoccupied safety test

- A. Lift armrests, swing footrests, and (if applicable) test the tilt function to verify safe and correct operation.

WARNING

Do not use the chair until all checks above have been completed. Re-tighten any loose hardware and correct any uneven leg heights before transferring a user. Failure to do so could cause an unstable seat and lead to a user fall or tip-over. This could result in serious injury or even death!

ADJUSTMENTS

ADJUSTING THE SEAT HEIGHT

The seat height can be adjusted by changing the position of the telescoping legs.

WARNING

Do not attempt to adjust the leg height while the user is seated in the chair. Doing so could cause the user to fall or chair to tip over. This could result in serious injury or even death!

1. Ensure the chair is on a stable, level surface.
2. Remove the user and any accessories from the chair before adjusting.
3. Press the brass button located on each inner leg and slide the inner leg section up or down to the desired height.
4. When the brass button aligns with a hole in the outer leg, it will "click" into place. Ensure each brass button is fully engaged and visible through the hole.
5. Repeat the process for all four legs, verifying that the chair remains level.

WARNING

If a brass button becomes stuck inside the inner leg, do not force it out. This could cause a collapse of the leg(s) and the user to fall or chair to tip over. This could result in serious injury or even death! Contact NuprodX, Inc. for assistance.

ADJUSTING THE FOOTREST HEIGHT

Adjustable Swing-Away Footrests

Height adjustments can be made by using the brass buttons or raising or lowering the footrest clamps on the legs. To adjust the footrest height via clamp:

1. Loosen the clamps to raise or lower the footrest tube.
2. Set the height so that the user's knees and hips are at approximately 90° when seated.
3. Ensure each footrest is set to the appropriate height for proper posture and balance.
4. Re-tighten the clamps to secure the footrests in place once the desired height is achieved.

Seat-Mounted Footrests

Slide up or down as required by the user's leg length using the brass buttons.

ADJUSTING THE CANTILEVER ARMREST HEIGHT & POSITION

The cantilever armrests can be adjusted to suit the user's comfort and transfer needs. Each armrest can be raised or lowered by repositioning the arm clamp along the backrest post.

1. Using the yellow T-handle Allen wrench, loosen the socket bolts located on each arm clamp.
2. Raise or lower the armrest to the desired height.
3. Once the correct height is achieved, re-tighten the socket bolts to securely lock the armrest in place.
4. For safety, the cantilever armrests lock automatically in the down position during use.
5. To lift the armrest for a side transfer or cleaning, push the armrest slightly toward the back of the chair to release the locking mechanism, then raise it upward into the open position.
6. When returning the armrest to the down position, ensure it clicks securely into the locked position before use.

WARNING

When folding or lowering the armrests, take care to keep hands, fingers, and clothing clear of the hinge and locking areas to prevent serious injury.

REMOVING THE ARMREST

If desired, the armrests can be removed for storage during traveling.

Cantilever Arms

For quick-release arms, simply press the quick-release button on each arm and pull outwards. For non-quick-release arms:

1. Unscrew the socket bolts that attach the arms to the backrest posts using the yellow T-handle Allen wrench.
2. Remove the arm assemblies from the posts and re-insert the socket bolts into the arm assemblies so they are not lost.
3. To re-install the cantilever arms, simply follow the steps above in reverse.

Swing-Away Arms

1. Press the release latch on the arm.
2. Lift up and pull the arm away from the chair.

CHANGING THE COMMODOE CUSHION ORIENTATION

If desired, the commode opening can be orientated to the left, right, or rear for 17x17 or 20x20 seat cushion sizes.

1. Unscrew the four cushion mounting screws on the underside of the cushion using the provided 5/32" Allen wrench.
2. Orient the cushion/commode slot to the desired orientation.
3. Reattach the cushion to the frame with the four mounting screws using the 5/32" Allen wrench.

CHANGING THE TILT CRANK SIDE

1. Remove the "crank handle" by removing the socket screw with an 1/8" Allen wrench.
2. Remove the cap from the opposite side of the frame, again using an 1/8" Allen wrench, and install it on this side.

3. Reinstall the crank handle on the opposite side of the frame.

CHANGING THE WHEEL ORIENTATION

RX MODELS ONLY

This operation is recommended only for chairs equipped with 20" rear wheels. If desired, the 20" wheels can be swapped with the front caster wheels so that the large wheels are positioned at the front of the chair.

1. Remove the backrest assembly from the chair.
2. Remove all four legs from the seat frame.
3. Detach the two casters and the footrest clamps from the front two 11-hole outer legs.
4. Remove both wheels from the axle blocks.
5. Remove the axle blocks from the two 10-hole offset outer legs and reinstall them onto the two 11-hole outer legs.
6. Reattach the footrest clamps to the 11-hole outer legs, positioning them underneath the axle blocks.
7. Reinstall the two casters into the 10-hole offset outer legs.
8. Reassemble the entire chair (see **SETTING UP**).

REMOVAL OF BIG WHEELS

SELF-PROPELLED MODELS ONLY

If desired, the big wheels can be removed for storage or cleaning.

1. Make sure the wheel locks are in the unlocked position.
2. Press down on the button of the quick release axle located in the center of the wheel.
3. With the quick release axle button depressed, pull the entire wheel assembly outward.
4. To re-install the wheels, simply follow the steps above in reverse.

WARNING

When reinstalling the quick release axle into the axle block, ensure that the detent balls are engaged by trying to pull out the axle. There should be a slight amount of back and forth movement. If the quick release axle is not reinstalled into the axle block correctly the whole wheel assembly could dislodge during usage. This could cause serious injury or even death!

INSTRUCTIONS FOR SAFE USE

1. Always engage wheel locks, including caster wheels and/or self-propelled wheels, before transferring the user onto or off of the chair, and whenever the chair is stationary.
2. Do not exceed the maximum user weight capacity specified for your model.
3. The MultiChair 4000 Series is intended for indoor use only.
4. Never lift or carry the chair while a user is seated.
5. Remove or swing away the footrests before performing any transfer.
6. For a side transfer, raise the armrest to provide clearance.
7. After transfer, return the armrest to its locked position and confirm that it is fully secured.

8. Replace footrests after transfer. Never stand on the footrests.

OPERATING INSTRUCTIONS

Before operating the MultiChair 4000 Series, ensure that the chair has been fully assembled and all components have been checked for stability and security. Refer to the **FINAL CHECK** section before first use.

⚠ WARNING

If thresholds, shower curbs or barriers are encountered that exceed ½" in height, the area in question should be brought within the code for disabled accessibility. Failure to do so could result in chair tip-over. This could result in serious injury or even death!

TILT MECHANISM

1. Pull the spring loaded crank lever outwards and rotating the handle.
2. Once the tilt angle is reached, we recommend either having the unit at its highest/largest tilt angle or flat, slowly release the spring loaded lever and it will automatically lock itself into one of the indexing holes on the indexing disk.
3. Index the handle by first positioning the seat in the horizontal position. The end of the crank handle should be approximately over the flat head socket bolt on the frame.

COMMODOE CHAIR

1. Position the MultiChair over the toilet, so that the center of the commode opening is directly above the center of the toilet bowl.
2. Adjust the height of the MultiChair accordingly by setting the position of the brass buttons on the four inner legs to the various adjustment holes on the outer legs.

⚠ WARNING

Ensure that the height of each inner leg is set to make the MultiChair level. If this is not properly done, the MultiChair could tip to one side causing the user to fall. This could result in serious injury or even death!

SHOWER CHAIR

1. Position the MultiChair inside the bathtub/shower.
2. Adjust the height of the MultiChair accordingly by setting the position of the brass buttons on the four inner legs to the various adjustment holes on the outer legs.

LECKEY CHAIR

Refer to Leckey Advanced Bath Chair user manual for operating instructions, including how to install and adjust positioning belts and how to adjust tilt settings.

TX PACKING INSTRUCTIONS

(SEE FIGURES A-D)

1. Arrange the four legs (with caster wheels attached) on the foam panel at the bottom of the carrying case and then

place the double sided foam panel on top of the legs.

2. Insert the rolled up cloth back into the cavity of the seat frame. Then place the folded seat frame on top of the double side foam panel.
3. Place the backrest posts with folded cantilever arms around the seat frame (shown here without the fabric socks). Then secure the legs, seat frame, backrest posts/arms using the black straps which are attached to the carrying case.
4. Place the folded footrests, tools, & instructions in the zipper pouch located in the front of the carrying case.

STORAGE & TRANSPORTATION

Always store the chair indoors in a dry, temperature-controlled environment.

Avoid exposure to moisture, direct sunlight, or extreme temperatures for long periods, as this may cause discoloration or degradation of plastic and padded components.

When transporting, remove all accessories and secure all loose parts to prevent damage.

If the chair is transported in a vehicle, ensure it is properly secured and supported to avoid impact or bending of the frame.

Before reuse, inspect all parts for proper fit and function.

CLEANING & MAINTENANCE INSTRUCTIONS

Inspect and tighten regularly

Periodically check all screws, bolts, clamps, and fasteners throughout the chair. Tighten as necessary to maintain stability and safety.

Inspect for wear or fatigue

Examine the frame, seat cushion, and hardware for signs of corrosion, cracks, or loose fittings. Replace any worn or damaged components immediately.

Clean routinely after use

Wipe down all aluminum, plastic, and padded surfaces with a soft cloth and mild soapy water. Rinse with clean water and dry thoroughly using a soft towel.

Avoid harsh chemicals

Do not use bleach, abrasive cleaners, or wire wool pads, as these may damage the finish or hardware.

Commode pan care

Remove and wash the commode pan with warm soapy water after each use. The commode pan must not be autoclaved or exposed to high heat.

Post-cleaning inspection

After cleaning or maintenance, always verify that all parts move freely and lock securely before returning the chair to service.

⚠ WARNING

The owner of this chair is responsible for ensuring that it has

been set up and adjusted by a trained service professional under the advice of a health care provider. The chair may require periodic safety and function checks or certain in-use adjustments that can be performed by the owner, caregiver, or authorized dealer if desired. Always use parts and/or accessories that have been recommended or approved by Nuprodx, Inc. when servicing this chair.

SAFETY CHECKLIST

The user or caregiver should perform these weekly and monthly checks to maintain the safety of the chair. If an item is not working properly, please contact your Nuprodx dealer.

Weekly	3 Months	6 Months	Annually	Safety & Function Check
☒				Visually check for loose hardware
		☒		Armrests
	☒			Casters
		☒		Frame
	☒			Upholstery
			☒	Service by Nuprodx dealer

LIFE EXPECTANCY

We estimate a life expectancy of 5 years for this product, provided that:

- It is used in strict accordance with the intended use as set out in this document.
- All service and maintenance requirements are met.

The estimated life expectancy can be exceeded if the product is carefully used and properly maintained. The life expectancy can also be considerably reduced by extreme or incorrect usage. The fact that we estimate a life expectancy for this product does not constitute an additional warranty

DISPOSAL

The product does not contain any hazardous substances. Dispose of the individual parts at your local facility

LIMITED WARRANTY

WARRANTY ACTIVATION: Please read this warranty before operating or using your MultiChair. To activate the warranty on your MultiChair register it online (www.nuprodx.com/form-registration/index.htm) or by phone. By operating or using the chair, you agree to the terms of this warranty.

WARRANTY: Nuprodx, Inc. warrants this product against defects in material and workmanship as follows: There is a 10-Day

conditional money-back warranty. During this time period, the customer is permitted to try out the MultiChair, fully-clothed, keeping the chair in "like new" condition (NOTE: "Like new" condition, in terms of the MultiChair, is defined by Nuprodx, Inc. as no visible wear/usage/water marks for the MultiChair to be accepted and refunds issued when returned to Nuprodx, Inc.). If the customer decides that the MultiChair will not work for them, before the 10-Day period has passed and with the authorization of Nuprodx, the chair can be returned for a full refund minus a 10% restocking charge (NOTE: The customer is responsible for both in-bound and out-bound freight). After the initial 10-Day period has passed, there is a two-year limited warranty for all parts of the chair, with the exception of the seat and back cushions. The seat and back cushions carry a one-year limited warranty. The warranty does not cover normal "wear and tear" from everyday use of the product and custom parts/custom cushions are also excluded from the warranty and cannot be returned for a refund under any circumstances. Please see section E WARRANTY LIMITATION AND EXCLUSIONS for more info on warranty exclusions.

The warranty period begins on the date you receive the chair. For warranty service, please contact Nuprodx, Inc. no later than one month following the applicable warranty term. The chair will be repaired or replaced at the discretion of Nuprodx, Inc. with no charges to you for parts and labor, provided you have proof of purchase and of purchase date.

DISCLAIMER: Except for the above warranty, and the acknowledgement by Nuprodx, Inc. that the chair, as manufactured by it, is fit for the general purpose for which most persons acquire a chair of its kind, Nuprodx, Inc. provides that you accept the chair as is, without warranties, either express or implied. Nuprodx, Inc. makes no warranty of fitness for your particular purpose and no warranty of merchantability beyond that already stated. No warranties extend beyond the duration of the express warranty stated above.

WARRANTY LIMITATIONS AND EXCLUSIONS: The only obligation of Nuprodx, Inc. is to provide the purchaser with free repair and replacement as described above. This exclusive warranty remedy will not have failed as long as Nuprodx, Inc. is willing and able to repair or replace as described, but if this remedy should be held to have failed, the only remaining warranty obligation of Nuprodx, Inc. shall be to refund the acts beyond the control of Nuprodx, Inc. The warranty does not cover normal "wear and tear" from everyday use of the product. Custom parts/custom cushions are not covered under the warranty.

This warranty gives you specific legal rights, and you may have other rights that may vary from state to state.

This warranty does not apply to problems arising from normal wear, improper operation, improper maintenance, improper storage or similar disclaimer of implied warranties, and some do not allow limitations on how long an implied warranty may last. Some do not allow exclusion or limitation of incidental or consequential damages. So the above limitations or exclusions may not apply to you.

RETURN INSTRUCTIONS: In the event that you need to return the Nuprodx MultiChair, please follow the instructions below:

1. Review the information contained in your warranty to see if this applies to you
2. Obtain a Return Authorization # from Nuprodx, Inc.
3. Re-package the entire chair and its contents in the original packaging and ship to the following address:

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©2025 Sunrise Medical (US) LLC
2. Demandez un numéro d'autorisation de retour à Nuprodx, Inc.
Nuprodx, Inc. / Attn: RMA# xxxx