JAY[®]

UNION CUSHION

JAY[®] Union[™] Cushion

IMPORTANT CONSUMER INFORMATION

NOTICE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

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Manuel d'instructions

NOTE: Check all parts for shipping damage. In case of damage DO NOT use. Contact Carrier/Sunrise for further instructions.

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY Union Cushion is appropriate. Cushions should only be installed by an authorized Sunrise Medical supplier.

NOTICE— READ BEFORE USE

A. CHOOSE THE RIGHT CUSHION

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY Cushion is appropriate for your specific needs. Cushions should only be installed by an authorized Sunrise Medical supplier.

B. CHECK SITTING PRESSURE OFTEN

Your JAY cushion was designed for sitting comfort and postural support. Users should inspect their skin daily for redness. The clinical indicator of tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.

C. REVIEW THIS MANUAL OFTEN

Before using this cushion, you and each person who may assist you, should read this entire Manual and make sure to follow all instructions. Review the Warnings often, until they are second nature to you.

D. WARNINGS

The word "WARNING" refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons.

E. GENERAL WARNINGS

⚠ WARNING

Do not install this equipment without first reading and understanding this manual. If you are unable to understand the Warnings, Cautions, and/or Instructions, contact a qualified clinician or supplier - otherwise injury or damage may occur.

⚠ WARNING

Hook and loop Velcro should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the Velcro® to hold the cushion in place.

⚠ WARNING

Installing a cushion on a wheelchair may effect the center of gravity of the wheelchair. Installed incorrectly, it may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess if the addition of anti-tips, and/or an amputee axle adapter brackets may be needed to help increase stability after adding a cushion.

⚠ WARNING

Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.

⚠ WARNING

Your JAY cushion is designed to help reduce pressure. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. The JAY cushion is not a substitute for good skin care including, proper diet, cleanliness, and regular pressure relief. Be sure that the Velcro is engaged and able to hold the cushion in place.

F. PROTECTING YOUR CUSHION

⚠ WARNING

Always avoid exposing your cushion to sharp objects, excessive heat or open flame, and prolonged exposure to environmental conditions like freezing temperatures and/or direct sunlight.

G. OBSTRUCTIONS

⚠ WARNING

Make sure that there are never any obstructions between you and your cushion. Any obstruction will reduce product effectiveness and/or produce unnecessary pressure points for the user.

H. COVER ORIENTATION

⚠ WARNING

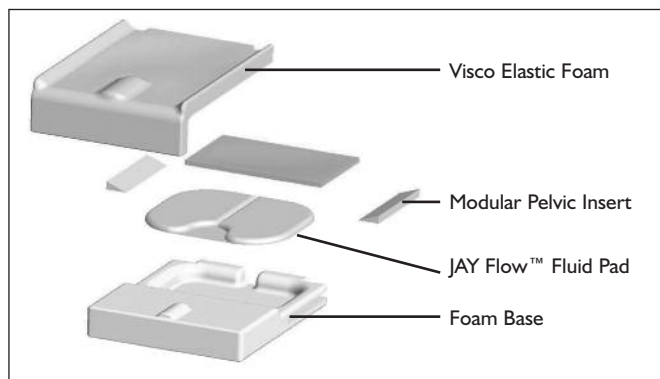
Product must be used with the cushion cover facing up. If the cover is not used correctly it may reduce or eliminate the cushion's benefits and could increase the risk of skin and soft tissue injuries.

MAXIMUM USER WEIGHT:

14" to 21" width = 350 lbs.

22" to 24" width = 600 lbs.

JAY UNION COMPONENTS

FITTING A PRESSURE DISTRIBUTING CUSHION

Select a cushion to match your hip width when in the seated position. The cushion length should extend to within 1" - 2" (2.5cm - 5cm) from the back of the knees. Place the cushion to the back of the wheelchair with the pelvic loading area toward the rear. When fit correctly, your hips should be to the back of the chair. Your ischials (seat bones) should be centered in the rear of the cushion. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. Adjust the footrests so your legs rest firmly but comfortably on the cushion.

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace stretched upholstery or purchase the JAY Adjustable Solid Seat.

Avoid using overly stretched backrest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched backrest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact an authorized supplier for a free demonstration of these products.

EASY MAINTENANCE & CLEANING

Regular cleaning and maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended.

Check the cover for tears and excessive wear and or any other abnormalities. Check the foam base to ensure foam consistency.

To clean cover

1. Remove the outer cover from the foam base and zip closed.
2. Machine wash in warm water and drip or tumble dry on low heat.

No ironing required.

NOTE: Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

Reinstall both covers over the foam base. Ensure the backs of the covers match up to the back of the foam base.

NOTE: Use only JAY covers on JAY fluid cushions.

To clean inner cover

Remove the cover and inner cover from the foam base. Wipe with warm water and soap. Wipe off with a clean cloth. Let it completely air-dry.

NOTE: Avoid submersion of the foam base in water.

Reinstall both covers over the foam base. Ensure the backs of the covers match up to the back of the foam base.

To clean foam base

Remove the inner and outer covers from the foam base. Wipe lightly with a damp cloth. Do not use soap. Do not submerge in water. Wipe off with a clean cloth. Let it completely air-dry before reassembly.

NOTE: Avoid submersion of the foam base in water.

Reinstall both covers over the foam base. Ensure the backs of the covers match up to the back of the foam base.

REASSEMBLING THE JAY UNION CUSHION AFTER CLEANING

Follow cleaning and maintenance instructions as described in "Easy maintenance and Cleaning" then reassemble as follows.

Insert the foam base into the inner cover with white label down and zipper to rear of cushion. Then insert base plus inner cover into the outer cover. Make sure white inner cover label is down and zipper is to the rear. Ensure that both zippers are at the rear of the foam base (the part number tag should be to the rear of the foam base) where the pelvis is typically located.

JAY UNION CUSHION WARRANTY

Every JAY Union cushion base is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable outer and inner covers are guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

For goods provided by Sunrise Medical Pty Ltd in Australia, our goods come with a guarantee by Sunrise that cannot be excluded under the Australian Consumer Law. You are entitled to replacement or refund for a major failure & for compensation for any other reasonable foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality & the failure does not amount to a major failure. The benefits to you given by this warranty are in addition to your other rights & remedies under a law in relation to the goods to which the warranty relates.

Record your serial number here for future reference:

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© 2025 Sunrise Medical (US) LLC
Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S
Sunrise Medical Canada Inc.