Owner's Manual

6000 Series

Mobile commode and bath/shower slider systems

IMPORTANT CONSUMER INFORMATION

- NOTICE:** This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.
- SUPPLIER:** This manual must be given to the user of this product.
- USER:** Before using this product, read this entire manual and save for future use.

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ATTENTION! PLEASE READ!

Please review the **LIMITED WARRANTY** included with these instructions. If the warranty terms are not acceptable, contact Nuprodx, Inc. for information about returning the chair for a refund (restocking fees and shipping charges will apply). This chair is designed and intended for use by a single occupant solely as an aid for showering or bathing, going to the toilet, or for indoor transfer. Any other use, including outdoor use, is strictly prohibited and will void the warranty.

OWNER'S MANUAL

6000

INDEX

INTRODUCTION	03
PARTS SUPPLIED	03
GENERAL WARNINGS	04
SETTING UP	04
FINAL CHECK	07
ADJUSTMENTS	07
INSTRUCTIONS FOR SAFE USE	09
OPERATING INSTRUCTIONS	09
TX PACKING INSTRUCTIONS	10
STORAGE & TRANSPORTATION	11
CLEANING & MAINTENANCE	11
SAFETY CHECKLIST	11
LIFE EXPECTANCY	11
DISPOSAL	11
LIMITED WARRANTY	11
FIGURES & NOTES	37-39

INTRODUCTION

The Nuprodx MultiChair 6000 Series is designed to provide safe, efficient, and highly versatile access for individuals who require assisted transfers into a bathtub or shower. This slider-based system combines a mobile rolling section with a stationary tub or shower base, allowing the user to move smoothly across a sliding bridge without the need for lifting. The modular design supports a wide range of clinical and home environments, helping caregivers manage bathing and toileting tasks with reduced strain and improved control.

Across the 6000 Series, the system accommodates users with varying postural, positioning, and support needs. Tilt models assist with pressure management and positioning, while non-tilt configurations provide a stable, straightforward transfer solution.

SERIAL NUMBER & MAX WEIGHT

The serial number label is located on the underside of the seat frame.

MC6000	300 lbs (136 kg)
MC6200	400 lbs (181 kg)
MC6000Tilt	300 lbs (136 kg)
MC6200Tilt	400 lbs (181 kg)
MC6000RS	300 lbs (136 kg)
MC6000RSTilt	300 lbs (136 kg)
MC6000TX	300 lbs (136 kg)
MC6000Leckey	160 lbs (73 kg)

PARTS SUPPLIED

SEAT FRAME

- (1) Outside frame (placed outside the tub or shower)
Includes commode seat (installed on tilt assembly if applicable) mounted to the shuttle frame
RS Models: includes commode seat (mounted to tilt assembly if applicable) installed on legs which are connected to the shuttle frame.
- (1) Inside frame (placed inside the tub or shower)

LEG ASSEMBLIES

Outside Frame

- Non-RS Models: (4) pinned offset straight legs with casters (5 or 6")
- RS Models: (4) 1.25" diameter plugs with crutch tips or casters (5 or 6") installed

Inside Frame

- Non-RS Models: (4) non-pinned offset straight legs with crutch tips
- RS Models: (4) 1.25" diameter plugs with crutch tips or casters (4") installed

BRIDGE ASSEMBLY

(1) connector bridge (used to connect the outside and inside frames during transfer)

SEAT ASSEMBLY

- (1) commode seat cushion mounted to shuttle frame
- (1) slider latch mechanism mounted underneath shuttle

frame (for locking and releasing seat during transfer)

BACK ASSEMBLY

- (2) backrest posts
- (1) backrest

Padded Backs: back will be installed on backrest posts

Cloth Backs: back included to wrap around backrest posts

- (2) armrest assemblies (left and right)

Cantilever Arms: arms will be installed on backrest posts

Swing-Away Arms: arms included to install on shuttle frame

- (1) headrest mounting bracket (if ordering tilt or headrest)

HEADREST ASSEMBLY (if ordering tilt or headrest)

- (1) headrest post
- (1) headrest pad

FOOTREST ASSEMBLIES

(2) footrest assemblies (left and right)

Adjustable Swing-Away Footrests

- (2) adjustable swing-away footrest assemblies (left and right)
- (2) footrest clamps (pre-mounted on front legs)

Seat-Mounted Footrests

(2) seat-mounted footrest assemblies (left and right)

HARDWARE & TOOLS

- (1) 1/4" yellow T-handled Allen wrench (included for assembly)
- (1) 5/32" Allen wrench (included for assembly)
- (1) 1/8" Allen wrench (included if ordering tilt or headrest)

LECKEY MODELS

- Leckey seat frame with mesh cover
- Leckey chest belt and hip positioning belt pre-installed

TRAVEL MODELS

(2) carrying cases with handle and casters

WARNING

The word "WARNING" refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons.

GENERAL WARNINGS

NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact Carrier/Nuprodx, Inc. for further instructions.

WARNING

DO NOT install this equipment without first reading and understanding the manual. If you are unable to understand the Warnings and Instructions, contact a qualified clinician or supplier - otherwise serious injury or damage may occur.

WARNING

Use only as intended. The MultiChair 6000 Series is designed and intended for use by a single occupant solely as an aid for showering or bathing, going to the toilet, or for indoor transfer. Any other use, including outdoor use, is strictly prohibited.

WARNING

A qualified attendant or caregiver must be present during all transfers and while the user is seated in the chair, unless the user has been specifically assessed as capable of independent operation.

WARNING

Hook and Loop Velcro should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the Velcro.

WARNING

Bolts, knobs, and quick-release pins should be tightened before each use. Loose hardware can cause instability or part failure.

WARNING

Only use Nuprodx-approved components and accessories. Unauthorized modifications may result in injury or void the product warranty.

WARNING

The chair is designed for wet environments, but water can make surfaces slippery. Always maintain control of the chair when near a bathtub, shower, or toilet.

WARNING

Examine the frame, casters, and locking mechanisms for wear, cracks, or corrosion. Discontinue use immediately if any damage is found and contact Nuprodx Inc. for service.

WARNING

Keep hands and clothing clear of pinch points when folding armrests, inserting legs into sockets, or adjusting moving parts.

SETTING UP

UNPACK & PREPARE

1. Remove all parts from the box.
2. Remove all tape and/or packaging from the parts. Keep shipping materials for future use and away from children and pets.
3. Lay out each component to ensure all parts are present.
4. Keep the yellow T-handle Allen wrench handy - it will be used for all tightening steps.

ASSEMBLE THE RS FRAME

1. Position the frames and install casters
 - A. Position the outside frame in front of the inside frame. The outside frame will be connected to the seat frame. The eyelet holes on each frame where the bridge will sit should face each other. **(SEE FIGURE 1)**
 - B. Install the casters onto both frames by sliding their 1.25" diameter plugs into the matching bored holes at the frame corners. The inside section utilizes the 4-inch casters, while the outside section utilizes the 5 or 6-inch casters. If ordering crutch tips for the inside frame, follow the same steps above.
 - C. Make sure each caster or crutch tip (if ordered) is fully seated at a 45-degree angle, then tighten the bolts at each associated frame corner using the ¼" yellow T-handle Allen wrench provided.

WARNING

All caster inserts and crutch tip inserts (if ordered) need to be properly fitted into their respective frame sockets and the associated pinch clamp bolts need to be securely tightened. Failure to do this creates a risk which could result in serious injury or even death!

2. Install the connector bridge
 - A. Position the connector bridge between both frames.
 - B. Set the four coned ends of the bridge into the mating eyelet features on the inside corners of each frame.
 - C. At each of the four bridge ends, slide the locking slide bars into the locked position to secure the bridge to both frames. **(SEE FIGURE 2)**
 - D. After locking the bridge, grab the black handle in the middle and lift up to ensure the bridge is properly locked.

WARNING

The connector bridge must be fully seated into the corresponding holes of both the outside section and inside section. If this is not done correctly, the shuttle/seat frame could dislodge itself from the base frames when transferring from one section to another. This could cause serious injury or even death!

3. Install the backrest (see backrest assembly instructions)
4. Install the armrests (see armrest assembly instructions)
5. Install the footrests (see footrest assembly instructions)

ASSEMBLE THE OUTSIDE FRAME

1. Identify the outside frame
 - A. This is the frame that includes the commode seat (with tilt system if applicable) attached to the shuttle frame.
 - B. Place the frame upside down on a protected surface to prevent cushion damage.

2. Install the (4) caster legs
 - A. Each leg has an offset elbow. The offset must face outward from the center of the frame. If ordering tilt, the rear, 90-degree bent legs will not have an offset elbow.
 - B. Identify the front legs. These have footrest clamps pre-installed if ordering adjustable swing-away footrests. The front legs must be installed on the front side of the outside frame.
 - C. Identify the rear legs. If ordering tilt or Leckey, these are the 90-degree bent legs.
 - D. Insert each leg into the corresponding socket until the stainless-steel pin clicks into the frame hole. There are 3 frame holes for installing straight legs at different angles (0, 45, and 90°). 45° is the ideal starting position. 0 and 90° may be used for increasing or decreasing the overall width. **(SEE FIGURE 3)**
 - E. The top end of each leg should fit easily into each socket of the frame.
3. Secure the legs
 - A. Using the yellow T-handle Allen wrench, tighten the corner bolts at all four mounting points.
 - B. Do not overtighten; bolts should be snug but not forcefully torqued.
4. Verify installation
 - A. Ensure each leg is fully seated, and all elbows angle evenly outward.
 - B. Once secure, flip the seat frame upright.
 - C. Gently rock the frame to confirm stability.

WARNING

If any of the legs are not fitted correctly into the socket of the seat frame or are not securely tightened there is a risk of the legs dislodging completely from the chair. This could cause serious injury or even death!

INSTALL THE LECKEY CHAIR

1. The front side of the Leckey frame has two mounting slots on each side, and the back side of the frame has one mounting slot on each side.
2. Pick up the Leckey chair and slide it in between the brackets of the Nuprodx frame. Be sure the mounting points align with the holes on the sides of the Leckey chair. **(SEE FIGURE 4)**
3. Use the provided hardware to secure the Leckey chair in its mounting brackets. Slide the bolt with the washer all the way through the bracket and chair from the outside.
4. Secure the bolt by threading the nut onto the opposite end of the bolt from the inside. **(SEE FIGURE 5)**
5. While holding onto the nut, use the yellow T-handle Allen wrench to tighten the chair down to the frame.
6. Repeat for all four corners.

INSTALL THE BACKREST ASSEMBLY

1. Identify the backrest assembly
This is the frame that includes 2 backrest posts with the backrest and armrests attached.
2. Insert the backrest posts

For Padded Backs: The backrest pad should face forward, toward the seat cushion and commode opening.

For Cloth Backs: The back posts should bend outwards,

away from the seat cushion and commode opening.

- A. Insert each backrest post into the rear mounting holes of the shuttle frame. **(SEE FIGURE 6)**
 - B. If backrest posts do not easily slide into the mounting holes, slightly loosen the internal clamp bolts on the bottom of each backrest post while inserting them into the mounting holes. Do not remove the internal clamp bolts completely.
 - C. Ensure both posts are fully inserted and the bottom indexing tab on the back-posts fit into the matching indexing features on the mating seat frame. The backrest will sit level.
 3. Tighten the clamps
 - A. Using the yellow T-handle Allen wrench, locate the internal clamp bolts on the bottom of the back posts and the rear corners of the shuttle frame.
 - B. Tighten each clamp evenly to secure the backrest posts firmly in place.
 - C. Avoid overtightening; ensure the posts remain vertical and aligned.
- For Padded Backs:** Also tighten the four button head capscrews that attach the back cushion to the back posts.
4. Verify installation
 - A. Gently push and pull on the backrest to ensure it does not move.
 - B. Confirm that both armrest clamps (if attached) remain level and secure.

WARNING

Both backrest posts must be completely engaged with the seat frame and securely tightened. If they are not fitted correctly, there is a risk of the posts dislodging from the seat frame. This could cause serious injury or even death!

INSTALL THE CLOTH BACK

1. Wrap the cloth back around the back posts and secure the 3 rear adjustable straps using the clips.
2. To prevent the cloth back from moving up and down the backposts, you may utilize the included Velcro strips by cutting the included Velcro strips to the desired length, attaching them to the back posts, and pressing the cloth back against the Velcro.

NOTE: Back tension adjustments are made possible by changing the length of the straps.

WARNING

Tension adjustments made to the cloth back should never be performed with the user in the chair. Make sure each strap on the cloth back is securely locked before use. Ignoring this safety precaution may cause the cloth back to come loose and possibly separate from the backrest posts, resulting in serious injury or even death!

INSTALL THE PUSH HANDLES

1. Position each push-handle at the top of each back post.
2. Press down on the brass buttons while sliding the push-handles into the back posts. **(SEE FIGURE 7)**

INSTALL THE HEADREST ASSEMBLY

1. The headrest clamp will be installed on the backrest assembly.

2. Use the yellow T-handle Allen wrench to loosen the two bolts on the headrest bracket.
3. Slide the extension tube with the headrest attached through the top of the headrest clamp.
4. Use the yellow T-handle Allen wrench to tighten the two bolts on the headrest bracket to secure the headrest in place.

INSTALL THE ARMREST ASSEMBLIES

Cantilever Armrests

These may come pre-installed on the backrest assembly. If not:

1. Remove the two bolts holding the armrest clamp together.
2. Take each arm with half of the clamp and position it onto the correct side of the corresponding back post to the desired height.
3. Take the second half of the clamp and position it inside the back post to secure the arm in place.
4. Tighten the armrest clamp bolts with the included yellow T-handle Allen wrench. **(SEE FIGURE 8)**

Swing-Away Armrests

1. Slip the end of the arm with the steel pin into the matching holes in the frame.
2. Lower down the arms until the latch engages around the black plastic ball attached to either side of the frame.

WARNING

The armrests are not designed to support the entire body weight of the user. Using the arms as a support for transferring into the chair and/or putting extreme amounts of pressure on them could result in damaging the arms or a fall and is not recommended. This could result in serious injury or even death!

INSTALL THE FOOTREST ASSEMBLIES

Adjustable Swing-Away Footrests

1. Locate the footrest components
 - A. Identify the left and right footrest assemblies.
 - B. Locate the footrest clamps pre-installed on the front legs of the outside frame. Verify that the clamps are positioned forward.
2. Install the footrests
 - A. Insert each footrest tube into its corresponding clamp on the front legs.
 - B. The footplates should face towards each other and level when installed correctly. **(SEE FIGURE 9)**
3. Adjust the clamp orientation and position by loosening the clamps and repositioning. Tighten the clamps after repositioning. Do not overtighten; bolts should be snug but not forcefully torqued.
4. Adjust the height of each footplate so it aligns with the user's leg position.

Seat-Mounted Footrests

1. Locate the footrest components
 - A. Identify the left and right footrest assemblies.
 - B. Locate the footrest mounting holes on the front corners of the shuttle frame.
2. Install the footrests
 - A. Slide the footrests into the front corners of the shuttle

frame and tighten the corner clamps. Do not overtighten; bolts should be snug but not forcefully torqued.

Verify Installation

Check that both footrests are firmly locked in position and do not rotate or slip when light downward pressure is applied.

WARNING

The footrests are not designed to support the entire body weight of the user. Using the footrests as a support for transferring into the chair and/or putting extreme amounts of pressure on them could result in damaging the footrests or a fall and is not recommended. This could result in serious injury or even death!

ASSEMBLE THE INSIDE FRAME

1. Identify the inside frame
 - A. This is the frame without a cushion attached.
 - B. Place it upside down on a soft, flat surface to prevent scratching or damage during assembly.
2. Install the legs
 - A. Locate the (4) non-pinned offset legs supplied for the inside frame.
 - B. Insert the legs into the four corner sockets of the frame.
 - C. Align the height adjustment holes so that they face outward, allowing easy access for height adjustments later.
3. Secure the legs
 - A. Using the yellow T-handle Allen wrench, tighten the corner bolts at all four leg connection points.
 - B. Tighten each bolt evenly and securely to prevent frame twisting.
 - C. Do not overtighten - bolts should be snug but allow the leg to remain properly seated.
4. Verify assembly
 - A. Confirm that all four legs are fully seated and evenly aligned.
 - B. Check that the frame rests level and stable on the floor surface.
 - C. Ensure that all adjustment holes are facing outward and accessible.
5. Flip the frame upright
 - A. Once verified, carefully turn the inside section upright and position it next to the outside section for bridge connection.

WARNING

Always adjust and secure the inside section legs before transferring the user or connecting the bridge. Failure to do so could cause the user to fall. This could result in serious injury or death!

WARNING

Improper leg alignment or uneven height may result in an unstable transfer surface causing the user to fall. This could result in serious injury or death!

CONNECT THE INSIDE & OUTSIDE FRAME

1. Position the frames
 - A. Place the outside section and inside section side by side, aligning them so that the bridge mounting points

face each other.

- B. Ensure both frames are on a flat, level surface.
2. Insert the connector bridge
 - A. Locate the connector bridge and align it with the mounting brackets on both frames.
 - B. Insert the four coned ends of the bridge into the mating eyelet features on the inside corners of each frame.
 - C. The bridge should rest evenly between both sections without gaps or tilting.
3. Level the frames
 - A. Check that the two frames are level and at equal height where they meet at the bridge.
 - B. If one frame sits higher or lower, adjust the leg height (using the brass button system) until both frames are perfectly aligned.
 - C. Verify that all legs rest evenly on the floor and that both frames are stable.
4. Verify bridge security
 - A. Ensure the bridge is firmly seated and cannot shift side to side.
 - B. Lock the bridge into place by sliding the tabs away from the center on all 4 corners of the bridge.

WARNING

Do not transfer a user or slide the seat at this stage. The bridge and frame alignment must be fully verified before operating the sliding mechanism. Failure to do so could cause the user to fall during transfer. This could result in serious injury or even death!

TEST THE SHUTTLE SEAT

1. Release the slider latch
 - A. Locate the slider latch on the underside of the shuttle frame.
 - B. Push the latch handle rearward to unlock the seat shuttle from the frame.
2. Slide the seat
 - A. Gently slide the seat along the bridge from the outside section to the inside section.
 - B. Movement should be smooth and free of resistance.
 - C. If the seat binds or stops midway, recheck bridge alignment and leg height.
3. Lock the seat
 - A. Once the seat reaches the inside section, the latch will automatically lock the seat in place. You should hear the seat latch locking.
 - B. Verify that the latch fully engages and is locked by trying to slide the seat.
4. Return the seat to the outside section
 - A. Unlock the latch again and slide the seat back across the bridge to the outside section.
 - B. Confirm the seat locks securely and sits squarely on the outside frame.
5. Verify function and stability
 - A. Repeat the slide motion several times to confirm consistent, secure operation.
 - B. Check that both frames remain level and the bridge stays seated throughout motion.

WARNING

Never attempt to operate or slide the seat while the user is seated until the bridge and latch operation have been fully tested and confirmed secure. Failure to do so could cause the user to fall. This could result in serious injury or even death!

FINAL CHECK

Before operating the MultiChair 6000 Series, carefully verify that the chair is fully assembled, properly adjusted, and safe for the intended user.

1. Verify all hardware and connections

- A. Ensure all bolts and clamps are tightened securely using the appropriate tools.
- B. Confirm that the brass height-adjustment button on each leg is fully engaged and visible.
- C. Make sure the connector bridge is fully seated in both frames with no gaps or movement.

2. Check frame stability & alignment

- A. Confirm that both frame sections rest evenly on the floor without rocking.
- B. Ensure the seat frame, backrest, and armrests are aligned and do not shift under gentle pressure.
- C. Verify that all leg heights remain consistent after adjustments.

3. Inspect functional components

- A. Slide the seat between the two sections to ensure smooth travel and secure locking at each end.
- B. Check that the seat cushion, armrests, and footrests are properly attached and firmly positioned.
- C. Move armrests and footrests through their full range to confirm unobstructed operation.
- D. Confirm the tilt mechanism locks safely before transferring a user (if equipped).

4. Perform an unoccupied safety test

- A. Operate the slider latch several times to confirm smooth engagement and secure locking.
- B. Lift armrests, swing footrests, and (if applicable) test the tilt function to verify safe and correct operation.

WARNING

Do not use the chair until all checks above have been completed. Re-tighten any loose hardware and correct any uneven leg heights before transferring a user. Failure to do so could cause an unstable seat and lead to a user fall or tip-over. This could result in serious injury or even death!

ADJUSTMENTS

ADJUSTING THE SEAT HEIGHT

The seat height can be adjusted by changing the position of the telescoping legs.

WARNING

Do not attempt to adjust the seat height while the user is seated in the chair. Doing so could cause the user to fall or chair to tip

over. This could result in serious injury or even death!

FOR NON-RS MODELS

1. Ensure the chair is on a stable, level surface.
2. Remove the user and any accessories from the chair before adjusting.
3. Press the brass button located on each inner leg and slide the inner leg section up or down to the desired height.
4. When the brass button aligns with a hole in the outer leg, it will “click” into place. Ensure each brass button is fully engaged and visible through the hole.
5. Repeat the process for all four legs, verifying that the chair remains level.

NOTE: Fine tune adjustments can be made by turning the crutch tips of the inside section.

FOR RS MODELS

1. Remove the backrest and loosen the (4) “pinch-clamp” corners of the seat frame with the ¼” yellow T-handle Allen wrench.
2. Remove the seat frame by lifting it straight up. Place the seat frame to the side.
3. Adjust the heights of each of the (4) legs by pressing in on the brass buttons and sliding the legs up-and-down to the desired height. Be sure that all the legs are at the same height.
4. Once the desired height is achieved, install the seat frame by lining up the leg holes at the bottom of the (4) corners of the seat frame with the legs. Slide the seat frame down onto the legs until they “bottom-out”.
5. Tighten the associated “pinch-clamp” corners with the ¼” yellow T-handle Allen wrench.
6. Place the backrest back into position and tighten associated bolts, again using the ¼” yellow T-handle Allen wrench.

WARNING

If a brass button becomes stuck inside the inner leg, do not force it out. This could cause a collapse of the leg(s) and the user to fall or chair to tip over. This could result in serious injury or even death! Contact NuprodX, Inc. for assistance.

ADJUSTING THE FOOTREST HEIGHT

Adjustable Swing-Away Footrests

Height adjustments can be made by using the brass buttons or raising or lowering the footrest clamps on the legs. To adjust the footrest height via clamp:

1. Loosen the clamps to raise or lower the footrest tube.
2. Set the height so that the user's knees and hips are at approximately 90° when seated.
3. Ensure each footrest is set to the appropriate height for proper posture and balance.
4. Re-tighten the clamps to secure the footrests in place once the desired height is achieved.

Seat-Mounted Footrests

Slide up or down as required by the user's leg length using the brass buttons.

ADJUSTING THE CANTILEVER ARMREST HEIGHT & POSITION

The cantilever armrests can be adjusted to suit the user's comfort and transfer needs. Each armrest can be raised or

lowered by repositioning the arm clamp along the backrest post.

1. Using the yellow T-handle Allen wrench, loosen the socket bolts located on each arm clamp.
2. Raise or lower the armrest to the desired height.
3. Once the correct height is achieved, re-tighten the socket bolts to securely lock the armrest in place.
4. For safety, the cantilever armrests lock automatically in the down position during use.
5. To lift the armrest for a side transfer or cleaning, push the armrest slightly toward the back of the chair to release the locking mechanism, then raise it upward into the open position.
6. When returning the armrest to the down position, ensure it clicks securely into the locked position before use.

WARNING

When folding or lowering the armrests, take care to keep hands, fingers, and clothing clear of the hinge and locking areas to prevent serious injury.

REMOVING THE ARMREST

If desired, the armrests can be removed for storage during traveling.

Cantilever Arms

For quick-release arms, simply press the quick-release button on each arm and pull outwards. For non-quick-release arms:

1. Unscrew the socket bolts that attach the arms to the backrest posts using the yellow T-handle Allen wrench.
2. Remove the arm assemblies from the posts and re-insert the socket bolts into the arm assemblies so they are not lost.
3. To re-install the cantilever arms, follow the steps above in reverse.

Swing-Away Arms

1. Press release latch on the arm.
2. Lift up and pull arm away from the chair.

CHANGING THE COMMODOE CUSHION ORIENTATION

If desired, the commode opening can be orientated to the left, right, or rear for 17x17 or 20x20 seat cushion sizes.

FOR TILT SLIDERS

1. Unscrew the four cushion mounting screws on the underside of the cushion using the provided 5/32” Allen wrench.
2. Orient the cushion/commode slot to the desired orientation.
3. Reattach the cushion to the frame with the four mounting screws using the 5/32” Allen wrench.

FOR NON-TILT SLIDERS

1. Unscrew the two rear cushion mounting screws on the underside of the cushion using the provided 5/32” Allen wrench.
2. Remove the slider latch by unscrewing the two mounting screws using the 5/32” Allen wrench.
3. Unscrew the two hexagonal standoffs that are attaching the cushion to the frame using a ½” open ended wrench or a small Crescent wrench.
4. Orient the cushion/commode slot to the desired orientation.
5. Reattach cushion using the two mounting screws in the

rear, the two hexagonal standoffs in the front, and then reattach the slider latch.

CHANGING THE TILT CRANK SIDE

1. Remove the “crank handle” by removing the socket screw with an 1/8” Allen wrench.
2. Remove the cap from the opposite side of the frame, again using an 1/8” Allen wrench, and install it on this side.
3. Reinstall the crank handle on the opposite side of the frame.

CHANGING THE SLIDE DIRECTION

The slide direction is defined by which direction the user, as seated in the sliding seat, will slide to get over the bathtub or shower stall. This desired starting slide direction is requested during the quotation and ordering process and the slider is built to the customer's request. However, sliding direction can easily be changed by following these simple steps:

NOTE: Do not attempt to unscrew the inner slotted screws of the release latch assembly. Doing so can cause the entire latch assembly to come apart, rendering it useless (There is no tool provided for this operation) .

1. Swap the outside section legs with the inside section legs in the following sequence.
 - A. Remove the front left leg on the outside section and remove the front left leg on the inside section and swap positions and retighten legs.
 - B. Remove the front right leg on the outside section and remove the front right leg on the inside section and swap positions and retighten legs.
 - C. Remove the rear left leg on the outside section and remove the rear left leg on the inside section and swap positions and retighten legs.
 - D. Remove the rear right leg on the outside section and remove the rear right leg on the inside section and swap positions and retighten legs.
2. Move the slider latch under the sliding seat to opposite side by unscrewing and removing the two mounting screws using the provided 5/32 Allen wrench. Once the slider latch has been positioned on the opposite side, mount using the two screws.
3. Move the latch stops on both the inside and outside sections by unscrewing both stops, single screw for each, with the 5/32 Allen wrench. Move to the opposite side and retighten. Note, there is a lock washer used on each stop.

At this point, the slide direction has been changed on all non-tilt models. There are additional steps for tilt:

TIPT ONLY

4. Move the tilt crank handle on the sliding seat assembly to the opposite side of the chair by loosening the screw on the pinch clamp portion of the crank handle with a 1/8” Allen wrench, pull the spring-loaded lever back, and then pull the crank handle outwards away from the chair. While doing this operation, put some weight in the seat so that it doesn't rock back with the crank handle removed.
5. Move the large white plastic washer to the opposite side of the chair.
6. Position the crank handle on the opposite side of the chair with the handle pointing forward at an angle of about 30 degrees down from horizontal.
7. Then pull the spring-loaded lever back and push up against plastic washer and retighten screw on the crank handle.
8. At this stage, the slide direction change has been completed. Perform an unoccupied test by releasing the

slider latch under the sliding seat, sliding the seat across to the opposite section, and ensure that the sliding seat “locks” on the latch stop.

INSTRUCTIONS FOR SAFE USE

1. Always engage the caster wheel locks before transferring the user onto or off of the chair, and whenever the chair is stationary.
2. Do not exceed the maximum user weight capacity specified for your model.
3. The MultiChair 6000 Series is intended for indoor use only.
4. Never lift or carry the chair while a user is seated.
5. Ensure the bridge is securely locked in place on both the inside and outside frames and both frames are level before sliding the seat between frames.
6. Remove or swing away the footrests before performing any transfer.
7. For a side transfer, raise the armrest to provide clearance.
8. After transfer, return the armrest to its locked position and confirm that it is fully secured.
9. Replace footrests after transfer. Never stand on the footrests.

OPERATING INSTRUCTIONS

Before operating the MultiChair 6000 Series, ensure that the chair has been fully assembled and all components have been checked for stability and security. Refer to the **FINAL CHECK** section before first use.

⚠ WARNING

If thresholds, shower curbs or barriers are encountered that exceed ½” in height, the area in question should be brought within the code for disabled accessibility. Failure to do so could result in chair tip-over. This could result in serious injury or even death!

1. Verify setup
 - A. Confirm all bolts, washers, and fasteners are tightened securely.
 - B. Verify the legs and frame connections are properly aligned and locked.
 - C. For tilt models, ensure the tilt assembly is locked and operates smoothly before proceeding.
 - D. Ensure the chair parts have been assembled correctly. Contact NuprodX, Inc. if unsure of anything.
2. Position the inside section
 - A. Place the inside section frame inside the bathtub or shower area in the desired position.
 - B. Ensure the surface beneath all four legs is clean, flat, and stable.
 - C. For tilt models, verify that the rear clearance allows full tilt range without obstruction.

3. Position the outside section
 - A. Move the outside section next to the bathtub or shower area, aligning it parallel with the inside section.
 - B. Engage the caster wheel locks to prevent movement during transfer.
4. Level both sections

Non-RS Models

- A. Adjust the height of the inside section using the brass buttons on the inner legs until it matches the height of the outside section.
- B. Fine-tune the height of the crutch tips on the inside section legs.
- C. Confirm that the bridge connection points between both sections are level and properly aligned.

RS Models

- A. Adjust the height of the inside section by adjusting the caster/crutch tip height. **(SEE FIGURE 10)**
 - B. Loosen the bolt with the ¼" Yellow T-handle Allen wrench to allow up-and-down adjustment of the caster/crutch tip.
 - C. Raise/lower the caster/crutch tip to desired height.
 - D. Tightly screw the bolt in place, locking the caster/crutch tip at desired height.
5. Stabilize the inside section
 - A. Rotate the elbow of each of the non-pinned legs on the inside section to adjust the width of the base to match the width of the bathtub.
 - B. Adjust the side-mounted suction cups outward to make contact with the both sides of the tub wall and lock into place.
 6. Install the connector bridge
 - A. Insert the connector bridge into the eyelet holes on both the outside and inside sections.
 - B. Push the bridge fully into place, ensuring all four shoulder screws or locking pins are seated securely.
 - C. Engage the bridge locks to the locked position before sliding the seat.
 7. Transfer the seat
 - A. Release the slider latch mechanism.
 - B. Gently slide the seat assembly from the outside section across the bridge and onto the inside section.
 - C. Ensure the latch re-engages and locks automatically once the seat reaches the inside section.
 - D. If the seat does not slide smoothly or lock properly, recheck the leg heights and bridge alignment before retrying.
 8. Verify stability
 - A. Once the seat is locked into the inside section, test the stability of the chair by gently rocking the frame.
 - B. The seat should remain locked and not lift or shift.
 - C. Remove the connector bridge to confirm the inside section remains stable on its own.
 - D. Reinsert the bridge and slide the seat back to the outside section for storage or repositioning.

⚠ WARNING

Never attempt to slide the seat while the user is seated unless the bridge is fully engaged and both sections are level. This could result in instability and user fall or tip-over. This could result in serious injury or even death!

⚠ WARNING

Always ensure the caster brakes are locked and the bridge is secure before any transfer. Failure to do so could result in user fall. This could result in serious injury or even death!

⚠ WARNING

For tilt models, ensure the tilt crank is locked and the seat is in a stable position before sliding or transferring the user. Failure to do so could result in instability and user fall or tip-over. This could result in serious injury or even death!

TILT MECHANISM

1. Pull the spring loaded crank lever outwards and rotating the handle.
2. Once the tilt angle is reached, we recommend either having the unit at its highest/largest tilt angle or flat, slowly release the spring loaded lever and it will automatically lock itself into one of the indexing holes on the indexing disk.
3. Index the handle by first positioning the seat in the horizontal position. The end of the crank handle should be approximately over the flat head socket bolt on the frame.

COMMODOE CHAIR

1. Position the MultiChair over the toilet, so that the center of the commode opening is directly above the center of the toilet bowl.
2. Adjust the height of the MultiChair accordingly by setting the position of the brass buttons on the four inner legs to the various adjustment holes on the outer legs.

⚠ WARNING

Ensure that the height of each inner leg is set to make the MultiChair level. If this is not properly done, the MultiChair could tip to one side causing the user to fall. This could result in serious injury or even death!

SHOWER CHAIR

1. Position the MultiChair inside the bathtub/shower.
2. Adjust the height of the MultiChair accordingly by setting the position of the brass buttons on the four inner legs to the various adjustment holes on the outer legs.

LECKEY CHAIR

Refer to Leckey Advanced Bath Chair user manual for operating instructions, including how to install and adjust positioning belts and how to adjust tilt settings.

TX PACKING INSTRUCTIONS

(SEE FIGURES A-I)

1. Drape the black strap across the bottom of the carrying case (not shown for clarity). Place the inverted shuttle & roll-in section frames inside the case.
2. Place the plastic panel over the inverted cushion and frame.
3. Place the foam ring on top of the frames.

- 4. Place the tub section frame on top of the foam ring.
- 5. Center the folded footrests down inside the frames.
- 6. Position the hourglass shaped foam piece on top of the folded footrest.
- 7. Place the legs with casters in the black plastic tray and center it over foam and footrests.
- 8. Place the connector bridge and foam on top of the legs with casters.
- 9. Wrap the black strap tightly around the completed package and position the backrest posts with folded cantilever arms and tub section legs next to it.

STORAGE & TRANSPORTATION

Always store the chair indoors in a dry, temperature-controlled environment.

Avoid exposure to moisture, direct sunlight, or extreme temperatures for long periods, as this may cause discoloration or degradation of plastic and foam components.

When transporting, remove accessories and secure all loose parts to prevent damage.

If the chair is transported in a vehicle, ensure it is properly secured and supported to avoid impact or bending of the frame.

Before reuse, inspect all parts for proper fit and function.

CLEANING & MAINTENANCE INSTRUCTIONS

Inspect and tighten regularly

Periodically check all screws, bolts, clamps, and fasteners throughout the chair. Tighten as necessary to maintain stability and safety.

Inspect for wear or fatigue

Examine the frame, seat cushion, and hardware for signs of corrosion, cracks, or loose fittings. Replace any worn or damaged components immediately.

Clean routinely after use

Wipe down all aluminum, plastic, and padded surfaces with a soft cloth and mild soapy water. Rinse with clean water and dry thoroughly using a soft towel.

Avoid harsh chemicals

Do not use bleach, abrasive cleaners, or wire wool pads, as these may damage the finish or hardware.

Commode pan care

Remove and wash the commode pan with warm soapy water after each use. The commode pan must not be autoclaved or exposed to high heat.

Post-cleaning inspection

After cleaning or maintenance, always verify that all parts move freely and lock securely before returning the chair to service.

⚠️ WARNING

The owner of this chair is responsible for ensuring that it has been set up and adjusted by a trained service professional under the advice of a health care provider. The chair may require periodic safety and function checks or certain in-use adjustments that can be performed by the owner, caregiver, or authorized dealer if desired. Always use parts and/or accessories that have been recommended or approved by Nuprodx, Inc. when servicing this chair.

SAFETY CHECKLIST

The user or caregiver should perform these weekly and monthly checks to maintain the safety of the chair. If an item is not working properly, please contact your Nuprodx dealer.

Weekly	3 Months	6 Months	Annually	Safety & Function Check
☒				Visually check for loose hardware
		☒		Armrests
	☒			Casters
		☒		Frame
	☒			Upholstery
			☒	Service by Nuprodx dealer

LIFE EXPECTANCY

We estimate a life expectancy of 5 years for this product, provided that:

- It is used in strict accordance with the intended use as set out in this document.
- All service and maintenance requirements are met.

The estimated life expectancy can be exceeded if the product is carefully used and properly maintained. The life expectancy can also be considerably reduced by extreme or incorrect usage. The fact that we estimate a life expectancy for this product does not constitute an additional warranty.

DISPOSAL

This product does not contain any hazardous substances. Dispose of the individual parts at your local facility.

LIMITED WARRANTY

WARRANTY ACTIVATION: Please read this warranty before operating or using your MultiChair. To activate the warranty on

your MultiChair register it online (www.nuprodx.com/form-registration/index.htm) or by phone. By operating or using the chair, you agree to the terms of this warranty.

WARRANTY: Nuprodx, Inc. warrants this product against defects in material and workmanship as follows: There is a 10-Day conditional money-back warranty. During this time period, the customer is permitted to try out the MultiChair, fully-clothed, keeping the chair in "like new" condition (NOTE: "Like new" condition, in terms of the MultiChair, is defined by Nuprodx, Inc. as no visible wear/usage/water marks for the MultiChair to be accepted and refunds issued when returned to Nuprodx, Inc.). If the customer decides that the MultiChair will not work for them, before the 10-Day period has passed and with the authorization of Nuprodx, the chair can be returned for a full refund minus a 10% restocking charge (NOTE: The customer is responsible for both in-bound and out-bound freight). After the initial 10-Day period has passed, there is a two-year limited warranty for all parts of the chair, with the exception of the seat and back cushions. The seat and back cushions carry a one-year limited warranty. The warranty does not cover normal "wear and tear" from everyday use of the product and custom parts/custom cushions are also excluded from the warranty and cannot be returned for a refund under any circumstances. Please see section E WARRANTY LIMITATION AND EXCLUSIONS for more info on warranty exclusions.

The warranty period begins on the date you receive the chair. For warranty service, please contact Nuprodx, Inc. no later than one month following the applicable warranty term. The chair will be repaired or replaced at the discretion of Nuprodx, Inc. with no charges to you for parts and labor, provided you have proof of purchase and of purchase date.

DISCLAIMER: Except for the above warranty, and the acknowledgement by Nuprodx, Inc. that the chair, as manufactured by it, is fit for the general purpose for which most persons acquire a chair of its kind, Nuprodx, Inc. provides that you accept the chair as is, without warranties, either express or implied. Nuprodx, Inc. makes no warranty of fitness for your particular purpose and no warranty of merchantability beyond that already stated. No warranties extend beyond the duration of the express warranty stated above.

WARRANTY LIMITATIONS AND EXCLUSIONS: The only obligation of Nuprodx, Inc. is to provide the purchaser with free repair and replacement as described above. This exclusive warranty remedy will not have failed as long as Nuprodx, Inc. is willing and able to repair or replace as described, but if this remedy should be held to have failed, the only remaining warranty obligation of Nuprodx, Inc. shall be to refund the acts beyond the control of Nuprodx, Inc. The warranty does not cover normal "wear and tear" from everyday use of the product. Custom parts/custom cushions are not covered under the warranty.

This warranty gives you specific legal rights, and you may have other rights that may vary from state to state.

This warranty does not apply to problems arising from normal wear, improper operation, improper maintenance, improper storage or similar disclaimer of implied warranties, and some do not allow limitations on how long an implied warranty may last. Some do not allow exclusion or limitation of incidental or consequential damages. So the above limitations or exclusions may not apply to you.

RETURN INSTRUCTIONS: In the event that you need to return the Nuprodx MultiChair, please follow the instructions below:

1. Review the information contained in your warranty to see if this applies to you.
2. Obtain a Return Authorization # from Nuprodx, Inc.

3. Re-package the entire chair and its contents in the original packaging and ship to the following address:

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©2025 Sunrise Medical (US) LLC
2. Demandez un numéro d'autorisation de retour à Nuprodx, Inc.
Nuprodx, Inc. / Attn: RMA# xxxx